

The Hearing Loss Help Company

COMPLETE KIT

Instructions for Use of Testing Stethoscope for all types of Hearing Aids

- A) When testing a body type hearing aid with air conduction receiver, remove the gray tubing and connector from the stethoscope and snap the receiver directly into the receptacle from which you removed the connector.**
- B) When placing the stethoscope on your head, make certain that the name (*Hal-Hen*) always faces away from your body...this is the only position in which the stethoscope will work.**
- C) When testing ear-level type hearing aids, including *in-the-ear, eyeglass, and behind-the-ear* types, you must have the gray tubing and connector attached to the stethoscope. You will note that a special bulb attachment has been installed at the end of the tubing. This attachment enables you to check the instruments without removing the earmold and tubing from the hearing aid. Merely place the earmold or canal tip into the opening of this special attachment and turn on the aid, adjusting the volume as desired. For in-the-ear type hearing aids, test in the same manner.**
- D) Also available in this kit is an extra large size clear fitting attachment. The standard tan colored attachment will in some cases not expand enough to grasp extra large size molds.**
- E) Also included in this kit is a canal insert needle attachment. This attachment will fit directly into all canal openings for direct sound pickup.**

Regards,

Curtis Dickinson. CEO

The Hearing Loss Help Company
www.Hearing-Loss-Help-Co.com
PH: 508 755 1120 FAX: 530 706 9997