

Power-Vac™

“Hearing Aid Cleaner”

Power-Vac is designed to help you maintain your hearing aids to peak performance by helping to loosen harmful earwax and environmental debris.

Cleaning Your Hearing Aid


1. Brush the entire surface of your hearing aid with your cleaning brush to remove loose particles. Be certain to pay special attention to the canal portion of the hearing aid or earmold.
2. Now use the Power-Vac to vacuum the entire surface of your hearing aid.
3. Place the canal portion of the hearing aid or earmold and slide it into the rubber tubing of the Power-Vac.
4. Slide the power switch on and allow the Power-Vac to run for approximately 5 minutes.
5. After sliding the power off remove the hearing aid or earmold from the tubing.
6. Brush the canal portion again this will help remove any debris that was loosened during the vacuum process.
7. Your hearing aid is now ready for use.

Your Power-Vac is powered by an A.C transformer that converts to 1½ Volts DC. Simply plug in the power cord to the Power-Vac and then plug in the transformer to a standard 110 volt outlet

To clean your Power-Vac follow the simple cleaning instructions, being sure that the Power-Vac is dry before attaching your hearing aid. **BE SURE TO PLACE ONLY THE HEAD IN WARM WATER.**

REMOVE THE RUBBER TUBING PRIOR TO CLEANING THE POWER-VAC.

Cleaning Instructions

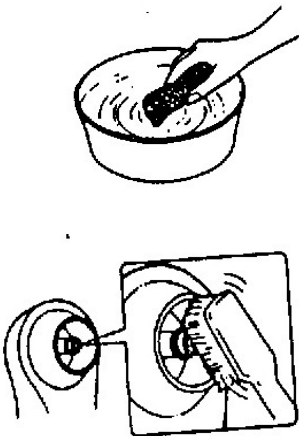

- A. Wash the head with warm water (below 60°C). Refer to diagram.
- B. Leave the power on to draw water into the plastic head for about 30 seconds. The flowing water cleans the device head automatically.
- C. Brush off any remaining substance with a toothbrush.
- D. Dry the device with tissue paper or cotton ball.
- E. Reattach the rubber tubing to the Power-Vac

It is important to see your hearing aid seller on a regular basis to insure peak performance from your hearing aids. Please keep in mind that your hearing needs may change over a period of time and that you should have your hearing tested on an annual basis.